

SOFTWARE DOWNLOAD & INSTALLATION

The **BODYSTAT for WINDOWS** programs have been designed to operate on **Windows** based operating system.


[To download the program:](#)

- The software can be downloaded directly from our website at the link below.
<https://download.bodystat.com>
- Select your device or required software from the dropdown menu and then enter the serial number of your Bodystat unit into the box provided.
- Click the "Download" button to download the software. The file you are downloading is called bodystat_installer.exe
- In addition, you will also need to download the activation number for your software by clicking "Certificate". instructions for activating the software number can be found under "Downloading with USB connection" in the software user manual.


[To install the program\(s\):](#)

- Run
- Double click the recently downloaded file Bodystat installer file (Bodystat_Installer.exe).
- The Bodystat software Installation Wizard will automatically begin.


The InstallShield Wizard will then appear and to install the software; follow the onscreen instructions.

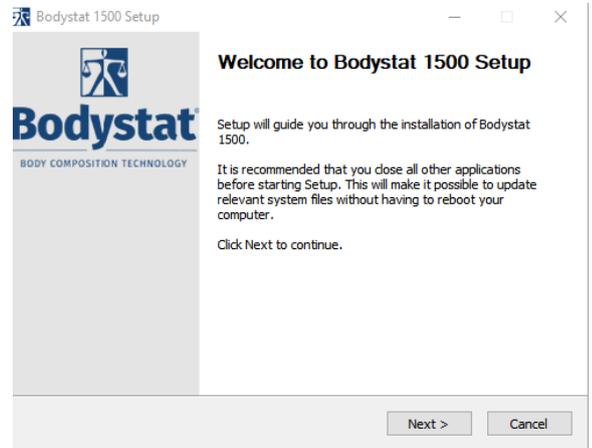


To install the program, the license agreement must first be accepted.

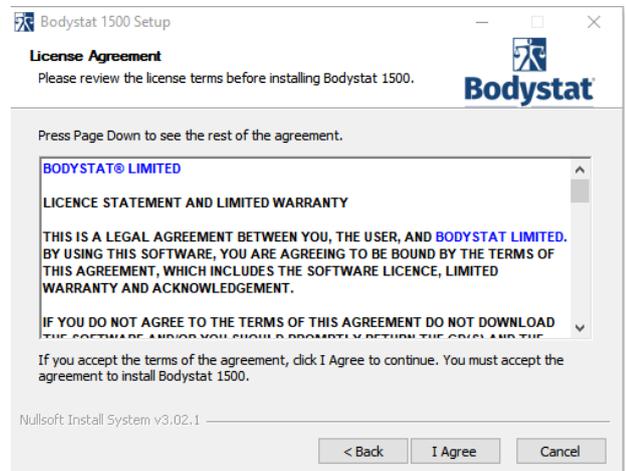

The software will automatically offer a destination for the program to be installed. If you would like to install into a different directory, click the **Change** button as shown in the diagram opposite.

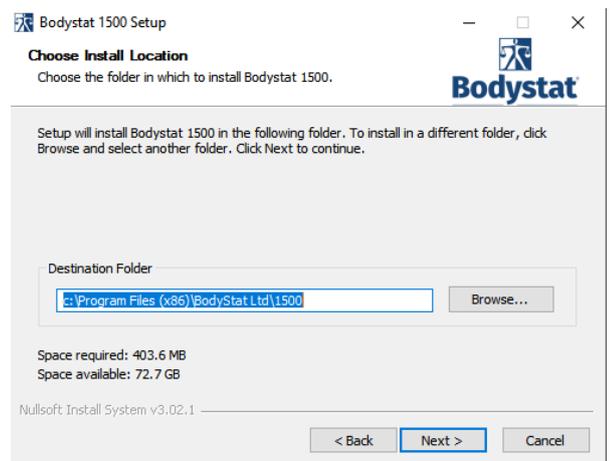


The software will create a shortcut on the start menu. If you do not require this, check the option at the bottom of the screen.

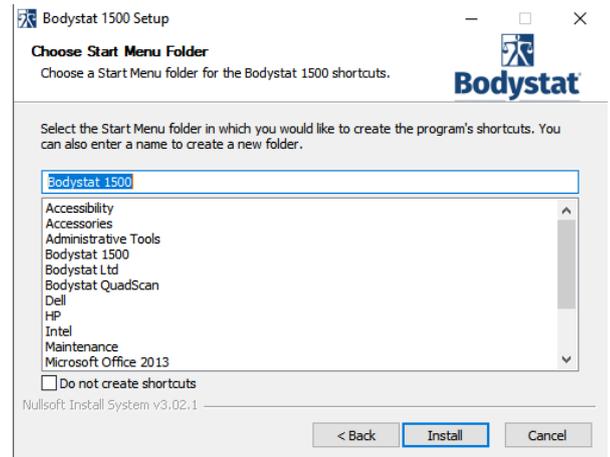


The software will also offer the option to install a shortcut on the desktop. To do this, tick the 'Start a shortcut' option.

Click 'Finish' to complete the download process. If successful, the software shall open automatically.

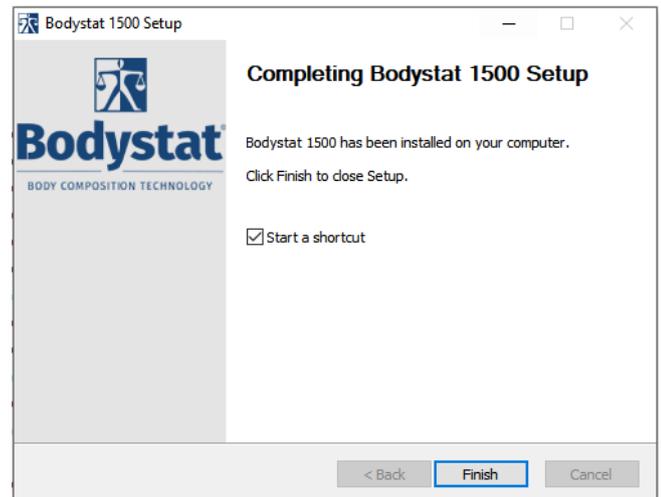

To connect your Bodystat device:

- ▽ GYVW hY 8Yj JW =bZc Vi Hrcb" Mci f'I G6 WUVY g\ci X'VY VcbBYWVX' VYhk YYb nci f'XYj JW UbX nci f'Vta di hYf'I G6 dcfh'
- ▽ 9bgj fY hUh hY XYj JW X]gd Uhg hY a YggU[Y''I G6 7cbbYVWVX'''' =Z hY XYj JW XcYg bchX]gd Umh]g'a YggU[Y'XYgd]h' hY WUVY VY]b['d'i [[YX]bz'i bd'i ['hY WUVY UbX hYb fY!]bgYfh'
- ▽ BYI hz WJW íG75B : CF 'I G6 89J =79Gí''
- ▽ Your device should be displayed with an option to click "Add Key". C]W í5XX?Ymí and you will be taken to the device key section of the settings page. Bck]bdi hnci f'8Yj JW bi a VYf'gdYVWVWgYf]U' _Ym hUh]g dfcj]XYX"

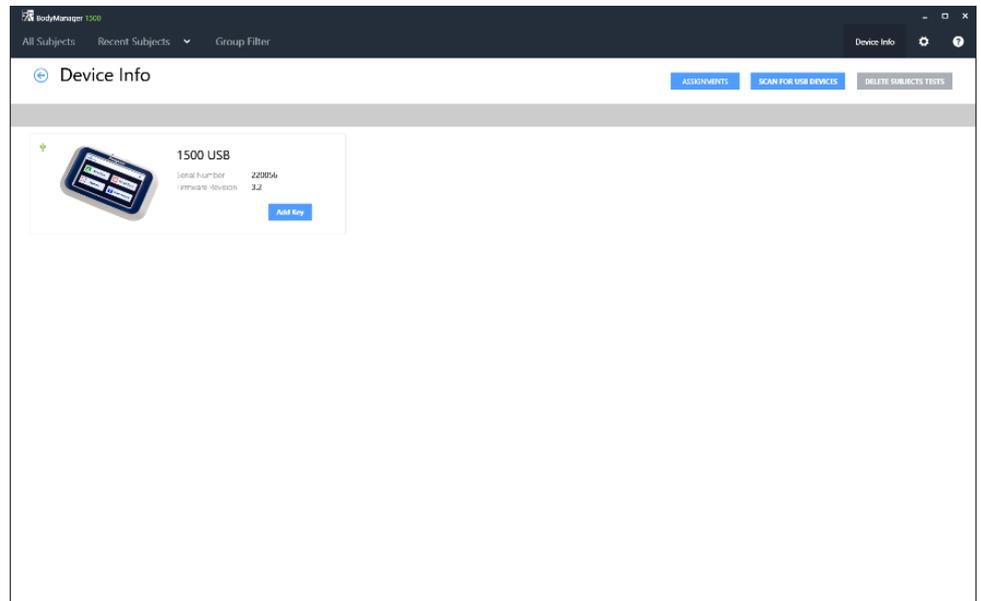

- Once you have entered your serial key, go back to the "Device Info" page and click "Scan for USB Devices" again. This time, the device should appear with an option to load tests.
- Data can now be download from the device. To allocate tests to subject files, select the assignments button from the device info page.